

GLA002 *Candida glabrata* Primer Set Instructions for Use

Product Overview	
Name	Candida glabrata Primer Set
Part Number	GLA002
Intended Use	The Candida glabrata Primer Set is an analyte specific reagent (ASR) for the detection of <i>Candida glabrata</i> DNA by polymerase chain reaction (PCR).
Regulatory Status	Analyte Specific Reagent. Analytical and performance characteristics are not established.
Number of vials	1
Fill Volume	200µL
Storage Temperature	-20°C or below
Number of acceptable of freeze thaws	24

Product Information	
Analyte Target	<i>Candida glabrata</i>
Fluorophore	ATTO 425
Sequence Target	ITS
Organisms determined to not have any binding affinity via <i>in silico</i> analysis	<i>Atopobium vaginae</i> , BVAB2, <i>Gardnerella vaginalis</i> , Megasphaera Type 1, Megasphaera Type 2
Handling	Always handle samples and product components with caution to avoid contamination. Protect product from prolonged exposure to light. Thaw completely, mix by pipetting or gentle vortexing and centrifuge briefly before use.
Precautions	Discard sample and product waste according to your local safety regulations.
Limitations	The product is for professional use only. Each laboratory must establish its own standard operating procedures and instructions.