

HHV6B002 HHV-6B Primer Set Instructions for Use

Product Overview	
Name	HHV-6B Primer Set
Part Number	HHV6B002
Intended Use	The HHV-6B Primer Set is an analyte specific reagent (ASR) for the detection of HHV-6B DNA by polymerase chain reaction (PCR).
Regulatory Status	Analyte Specific Reagent. Analytical and performance characteristics are not established.
Number of vials	1
Fill Volume	300µL
Storage Temperature	-20°C or below
Number of acceptable of freeze thaws	24

Product Information	
Analyte Target	HHV-6B
Fluorophore	FAM
Sequence Target	<i>U57</i>
Organisms determined to not have any binding affinity	Adenovirus, BK Virus, CMV, EBV, HHV-6A, HSV, JC Virus, and VZV
Handling	Always handle samples and product components with caution to avoid contamination. Protect product from prolonged exposure to light. Thaw completely, mix by pipetting or gentle vortexing and centrifuge briefly before use.
Precautions	Discard sample and product waste according to your local safety regulations.
Limitations	The product is for professional use only. Each laboratory must establish its own standard operating procedures and instructions.