

Product Overview	
Name	HPV Genotype 45 Primer Set
Part Number	HPV45002
Intended Use	The HPV Genotype 45 Primer Set is an analyte specific reagent (ASR) for the detection of HPV Genotype 45 DNA by polymerase chain reaction (PCR).
Regulatory Status	Analyte Specific Reagent. Analytical and performance characteristics are not established.
Number of vials	1
Fill Volume	150µL
Storage Temperature	-20°C or below
Number of acceptable of freeze thaws	24

Product Information	
Analyte Target	HPV Genotype 45
Fluorophore	ATTO 425
Sequence Target	E6/E7
Organisms determined to not have any binding affinity	HPV Genotypes: 6, 11, 16, 18, 26, 31, 33, 35, 39, 42, 43, 44, 51, 52, 53, 56, 58, 59, 66, 68, 73, 81, 82, 83
Handling	Always handle samples and product components with caution to avoid contamination. Protect product from prolonged exposure to light. Thaw completely, mix by pipetting or gentle vortexing and centrifuge briefly before use.
Precautions	Discard sample and product waste according to your local safety regulations.
Limitations	The product is for professional use only. Each laboratory must establish its own standard operating procedures and instructions.