

## MIY002 *Borrelia miyamotoi* Primer Set Instructions for Use

<b>Product Overview</b>	
Name	Borrelia miyamotoi Primer Set
Part Number	MIY002
Intended Use	The <i>Borrelia miyamotoi</i> Primer Set is an analyte specific reagent (ASR) for the detection of <i>Borrelia miyamotoi</i> DNA by polymerase chain reaction (PCR).
Regulatory Status	Analyte Specific Reagent. Analytical and performance characteristics are not established.
Number of vials	1
Fill Volume	150µL
Storage Temperature	-20°C or below
Number of acceptable of freeze thaws	24

<b>Product Information</b>	
Analyte Target	<i>Borrelia miyamotoi</i>
Fluorophore	CAL Fluor 635
Sequence Target	<i>glpQ</i>
Organisms determined to not have any binding affinity via <i>in silico</i> analysis	<i>Anaplasma phagocytophilum, Babesia, Borrelia burgdorferi, Ehrlichia</i>
Handling	Always handle samples and product components with caution to avoid contamination. Protect product from prolonged exposure to light. Thaw completely, mix by pipetting or gentle vortexing and centrifuge briefly before use.
Precautions	Discard sample and product waste according to your local safety regulations.
Limitations	The product is for professional use only. Each laboratory must establish its own standard operating procedures and instructions.