

VZV003 VZV Primer Set Instructions for Use

Product Overview	
Name	VZV Primer Set
Part Number	VZV003
Intended Use	The VZV Primer Set is an analyte specific reagent (ASR) for the detection of VZV DNA by polymerase chain reaction (PCR).
Regulatory Status	Analyte Specific Reagent. Analytical and performance characteristics are not established.
Number of vials	1
Fill Volume	200µL
Storage Temperature	-20°C or below
Number of acceptable of freeze thaws	24

Product Information	
Analyte Target	VZV
Fluorophore	CY5
Sequence Target	ORF62
Organisms determined to not have any binding affinity via <i>in silico</i> analysis	Adenovirus, BK Virus, CMV, EBV, HHV-6, JC Virus, HSV-1, and HSV-2
Handling	Always handle samples and product components with caution to avoid contamination. Protect product from prolonged exposure to light. Thaw completely, mix by pipetting or gentle vortexing and centrifuge briefly before use.
Precautions	Discard sample and product waste according to your local safety regulations.
Limitations	The product is for professional use only. Each laboratory must establish its own standard operating procedures and instructions.